



- 1 Remove paper from the back of the Wear&Care Strip
  - 2 Place top of Wear&Care Strip and fold around pad
  - 3 Stick the Wear&Care strip on the pad
- Note: Avoid opening the incontinence pad**



- 1 Open: Press the edge and end of the Wear&Care Sensor
  - 2 Place: Press Wear&Care Strip into the two holes first
  - 3 Close: Press the Wear&Care Sensor together and lock
- Note: Green light means Sensor on Strip is correctly seated**

Icon	Meaning
	<i>New pad registered. System OK</i>
	<i>First urine registered. No action required</i>
	<i>Full pad registered. Consider a pad change</i>
	<i>Maximum wear time achieved. Consider pad change</i>
	<i>Resident without Sensor.</i> <b>NOTE: No registration of pad status</b>
	<i>Action required.</i> See gray icon and text on screen for action
	<i>Low battery. Schedule charging of the Sensor</i>

### Note:

- » Updating the screen with good mobile coverage typically takes ½ - 2 minutes
- » Use only approved (set-up) pads, both night and day
- » If appears on the screen and the nappy is dry, it may be due to compression of the nappy when urinating
- » If is displayed on the screen and the diaper is full/leaking, it may be due to a delay in mobile signal or the diaper was not put on correctly
- » Red light on Wear&Care Sensor when closed: Charge Sensor
- » If still on screen after green light on Sensor, check lid / closing mechanism

